Organ Recovery  
systems

## LifePort<sup>®</sup> Kidney Transporter Cover

### Instructions for Use

#### Chicago

One Pierce Place, Suite 475W  
Itasca, IL 60143  
USA

T +1.847.824.2600

F +1.847.824.0234

Perfusion +1.866.682.4800

Helpline

#### Brussels

Da Vincilaan 2 Box 6  
1935 Zaventem  
Belgium

T +32.2.715.0000

F +32.2.715.0009

Perfusion +32.2.715.0005

Helpline

#### São Paulo

200 Josefina Street, Suite 215  
Guarulhos, SP 07093-080  
Brazil

T +55.11.3586.6259

F +55.11.3586.4944

Perfusion 0800.940.0057

Helpline

Instructions for Use printed in:

**INDICATIONS FOR USE**

The LifePort Kidney Transporter Cover is intended to assist in the transportation of the LifePort Kidney Transporter, and optional Biological Substance Category B (UN 3373) clinical samples and donor/organ documentation storage.

1. Check the Ice Container temperature and battery level before placing the LifePort Kidney Transporter inside the device compartment. Replenish ice and charge batteries if necessary before shipping.
2. Place the donor/organ documentation into the designated optional pouch. If required, use zip-tie to seal donor/organ documentation pouch for secured access.
3. With the Cover on the LifePort, place the Transporter inside device compartment. Ensure the front panel of the LifePort Kidney Transporter faces the transparent window on the front of the Cover.
4. Close the Cover using the zippers. If required, to secure the closed Cover, secure both zippers together with a zip-tie.
5. The shipper is responsible for the proper packaging of Biological Substance Category B (UN 3373) if biological samples will accompany the organ. All shipments of Biological Substance Category B (UN 3373) must comply with all applicable local, state, federal and country specific laws governing packing, marking and labeling.

---

**WARNING:** Blood, urine, fluids and other specimens containing or suspected of containing infectious substances must be shipped according to applicable government and International Air Transport Association (IATA) regulations. For more information consult your internal policies and procedure on biological sample shipments.

---

If required, secure this compartment using a zip-tie.

6. Store the Power Cord and extra batteries in either the left or right side compartment located on the upper portion of the LifePort Kidney Transporter Cover.
7. Whenever a power supply is available, the LifePort unit should be plugged into main power supply to maintain a full charge. You may gain access to the Battery Compartment of the LifePort Kidney Transporter Cover by opening the zipper located at the rear of the Cover.
8. Use either the left or right compartment located on the upper portion of the LifePort Kidney Transporter Cover to store the retractable handle.
9. During transportation, the LifePort Kidney Transporter Cover may be pulled by putting the three (3) components of the retractable handle together and hooking up the handle onto the two metal D-shape rings.
10. The handle can be kept in upright position by opening and tightening the designated strap on the side.
11. Shipping documents and Human Organ leaflet may be placed into the transparent pouches on top and at the rear side of the LifePort Kidney Transporter Cover.

**Explanation of Symbols**

Warning/Caution

Manufacturer

Lot Number

Date of Manufacturer, YYYY/MM

Sealable zipper for LifePort compartment

Human organ leaflet compartment

LifePort front panel window

Retractable handle storage compartment

Power cable and extra battery storage compartment

Access batteries and power

Optional Sealable Biological Substance Category B (UN 3373) compartment

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