KPS-1[®]

Kidney Perfusion Solution

Instructions for Use

 **Organ Recovery Systems, Inc.**
One Pierce Place, Suite 475W
Itasca, IL 60143
USA

T +1.847.824.2600
F +1.847.824.0234
Perfusion Helpline
+1.866.682.4800

Organ Recovery Systems NV
Da Vincilaan 2, Box 6
1831 Diegem
Belgium

T +32.2.715.0000
F +32.2.715.0009
Perfusion Helpline
+32.2.715.0005

ORS Representacoes do Brasil Ltda.
170 Moema Avenue, Suite 11 & 12
Sao Paulo, SP 04077-020
Brazil

T +55.11.3586.6259
F +55.11.3586.4944
Perfusion Helpline
+55.11.98638.0086

KPS-1

Kidney Perfusion Solution

Instructions for Use **en**

INDICATIONS FOR USE

KPS-1, Kidney Perfusion Solution is intended to be used for flushing and continuous hypothermic machine perfusion of kidneys at the time of their removal from the donor in preparation for storage, transportation and eventual transplantation into a recipient.

DEVICE DESCRIPTION

KPS-1, Kidney Perfusion Solution (having the same composition as UW Machine Perfusion Solution) is a clear, sterile, non-pyrogenic, non-toxic solution for the in-vitro flushing and temporary continuous perfusion preservation of explanted kidneys. This solution has an osmolality of 300 ± 15 mOsm/kg, a sodium concentration of 100 mEq/L, a potassium concentration of 25 mEq/L, and a pH of 7.4 ± 0.1 at room temperature. Based on the sodium/potassium ratio, the composition is thus consistent with that of an extracellular solution.

The perfusate should be cooled to approximately 2° to 8°C (36° to 46°F) prior to use and should be used in a perfusion machine that is capable of maintaining temperature within the above specified range.

SUGGESTED VOLUME

The recommended perfusate volume is 1000 mL for one human kidney.

STORAGE

KPS-1 may be stored at room temperature and cooled before use. Do not freeze or expose to excessive heat.

CAUTION: The solution should be stored indoors in a dry location out of direct sunlight.

PREPARATION

Remove clear overwrap prior to use. Carefully open the clear pouch being careful not to damage or tear the interior perfusate bag. If the interior perfusate bag is damaged during opening process, discard solution.

Inspect to ensure there is no visible contamination, particulate matter or precipitates in the solution or ports. If the solution is clear and no contamination, particulates or precipitates are observed, the solution is safe to use.

CAUTION: Do not use if there is evidence of contamination, particulates or precipitates in the solution or the ports. Contact Organ Recovery Systems to make arrangements to return solution.

After pre-cooling the kidney by vascular flushout using KPS-1, Kidney Perfusion Solution or other suitable cooled solutions (SPS-1[®], Ringer's, or saline), the kidney can be placed on a suitable perfusion apparatus and machine perfused according to the manufacturer's (or perfusion center's) protocol.

CONTRAINDICATIONS: There are no known contraindications when used as directed.

WARNINGS: Not for Direct Intravenous Infusion.

WARNINGS: For single use only. Do not reuse, reprocess or resterilize. Reusing, reprocessing or resterilization of single-use devices creates a potential risk of patient or user infections due to contamination. This contamination may lead to injury, illness or other serious patient complications. Discard any unused portion.

PRECAUTIONS

KPS-1, Kidney Perfusion Solution includes constituents (Hydroxyethyl starch [HES]), which have caused hypersensitivity reactions in some patients. Physicians should be alert to treat possible reactions.

ADVERSE REACTIONS

No adverse reactions thought to be attributable to the perfusion solution have been observed when the solution is used as described.

This page intentionally left blank.

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SOLUTION COMPOSITION

Constituents	Amount/1000 mL	Concentration, mM
Calcium chloride (dihydrate)	0.068 g	0.5
Sodium hydroxide	0.70 g	
HEPES (free acid)	2.38 g	10
Potassium phosphate (monobasic)	3.4 g	25
Mannitol (USP)	5.4 g	30
Glucose, beta D (+)	1.80 g	10
Sodium gluconate	17.45 g	80
Magnesium gluconate D (-) gluconic acid, hemimagnesium salt	1.13 g	5
Ribose, D (-)	0.75 g	5
Hydroxyethyl starch (HES)	50.0 g	n/a
Glutathione (reduced form)	0.92 g	3
Adenine (free base)	0.68 g	5
Sterile water for injection (SWI)	To 1000 mL volume	n/a

This page intentionally left blank.

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SHELF LIFE

The shelf life for an unopened bag is 2 years.

Solution should be used immediately after opening in accordance with specific perfusion center protocol.

LIFETIME

Discard solution after each case, or at least after 23 hours.

Rx Only

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

CAUTION: Do not use if frozen, cloudy or exposed to extreme heat.

*University of Wisconsin[®] is a registered trademark of the Board of Regents of the University of Wisconsin System. Organ Recovery Systems, Inc. has no affiliation with the Board of Regents of the University of Wisconsin System.

Explanation of Symbols

STERILE A

Sterile Medical Devices
Using Aseptic Technique
(Aseptic Fill)

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