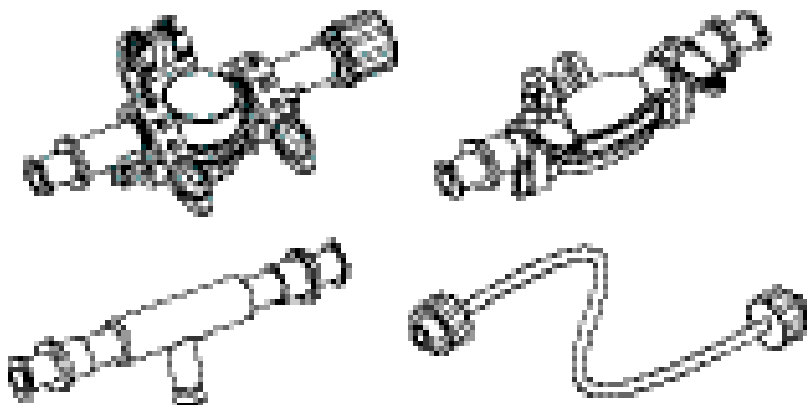
# Organ Recovery systems

## LifePort® Kidney Transporter Disposable Cannula

### Instructions for Use

Universal SealRing®, SealRing®, Straight, and Coupler  
are for use only with LifePort Kidney Transporter system.  
Patent Pending.

 **Organ Recovery Systems, Inc**  
One Pierce Place, Suite 475W  
Itasca, IL 60143  
USA

T +1.847.824.2600  
F +1.847.824.0234  
**Perfusion Helpline**  
+1.866.682.4800

**Organ Recovery Systems NV**  
Da Vincilaan 2 Box 6  
1831 Diegem  
Belgium

T +32.2.715.0000  
F +32.2.715.0009  
**Perfusion Helpline**  
+32.2.715.0005

**ORS Representacoes do Brasil Ltda.**  
170 Moema Avenue, Suite 11 & 12  
São Paulo, SP 04077-020  
Brazil

T +55.11.3586.6259  
F +55.11.3586.4944  
**Perfusion Helpline**  
+55 11 98638 0086

Not all products pictured are commercially available in all markets.

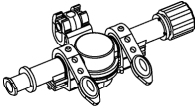
Instructions for Use printed in:

[www.organ-recovery.com](http://www.organ-recovery.com)

[www.patents-organrecoverysystems.com](http://www.patents-organrecoverysystems.com)

## Universal SealRing

UCAN0003  
UCAN0005  
UCAN0007  
UCAN0009

### INDICATIONS FOR USE

LifePort Kidney Transporter (LKT) system is intended to be used for the continuous hypothermic machine perfusion of kidneys for the preservation, transportation and eventual transplantation into a recipient.

### DEVICE DESCRIPTION

LifePort Kidney Transporter Disposable Universal SealRing Cannula is used when the vessel to be perfused ends with or without an aortic patch or similar condition.

The complete LifePort Kidney Transporter system is comprised of the following components:

- LifePort Kidney Transporter (LKT100P/LKT101P)
- LifePort Kidney Transporter Disposable Perfusion Circuit (LKT201)
- LifePort Kidney Transporter Disposable Sterile Drape (LKT300)
- LifePort Kidney Transporter Disposable Cannula (CAN/UCAN)

### INSTRUCTIONS FOR USE

Prepare LifePort Kidney Transporter and LifePort Kidney Transporter Perfusion Circuit as described in their Instructions for Use. If cannulating multiple vessels, refer to LifePort Kidney Transporter Disposable Coupler Instructions for Use.

**CAUTION:** Where noted, perform the following procedure on an aseptic field using aseptic technique.

1. *Using standard aseptic technique*, introduce LifePort Kidney Transporter Disposable Universal SealRing Cannula onto the sterile field.
2. *Using standard aseptic technique*, open LifePort Kidney Transporter Disposable Universal SealRing Cannula by unhooking the straps, unsnapping and opening the right anvil.
3. *Using standard aseptic technique*, position the vessel in the center of the anvils so the terminated end of the vessel is approximately 1.5–2.0mm above the top surface of the anvils.
4. *Using standard aseptic technique*, close the right anvil, capturing the vessel with an audible click.

**NOTE:** If necessary, retention stays can be added to the vessel using suture or other appropriate material. Suture Tie Down Cleats and slots have been provided to facilitate positioning.

5. *Using standard aseptic technique*, lower the upper portion of LifePort Kidney Transporter Disposable Universal SealRing Cannula while carefully bringing the conical seal into the interior of the vessel.

**NOTE:** Viewing through the magnified sight-glass, verify that the vessel orifice is centered, circular in shape and not occluded.

6. *Using standard aseptic technique*, wrap each strap around both cannula halves and secure the straps to their posts.
7. *Using standard aseptic technique*, place the kidney in the Kidney Cradle and snap LifePort Kidney Transporter Disposable Universal SealRing Cannula into the Cannula Mount.

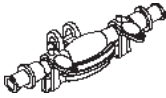
**NOTE:** If perfusing multiple vessels, only the main vessel cannula shall be connected to the Cannula Mount.

8. *Using standard aseptic technique*, adjust the height of the Cannula Mount and rotation of the cannula to comfortably position the vessel.
9. *Using standard aseptic technique*, visually inspect the vessel, ensuring there are no twists or occlusions.
10. *Using standard aseptic technique*, secure the Mesh Organ Restraint over the kidney in the Kidney Cradle, allowing for slight swelling while being perfused.

**NOTE:** Follow the procedure in the LifePort Kidney Transporter Disposable Perfusion Circuit Instructions for Use on how to place the kidney and Kidney Cradle within LifePort Kidney Transporter Disposable Perfusion Circuit.

## SealRing

CAN0720  
CAN1035

### INDICATIONS FOR USE

LifePort Kidney Transporter (LKT) system is intended to be used for the continuous hypothermic machine perfusion of kidneys for the preservation, transportation and eventual transplantation into a recipient.

### DEVICE DESCRIPTION

LifePort Kidney Transporter Disposable SealRing Cannula is used on explanted kidneys when the vessel to be perfused ends with an aortic patch or similar condition.

The complete LifePort Kidney Transporter system is comprised of the following components:

- LifePort Kidney Transporter (LKT100P/LKT101P)
- LifePort Kidney Transporter Disposable Perfusion Circuit (LKT201)
- LifePort Kidney Transporter Disposable Sterile Drape (LKT300)
- LifePort Kidney Transporter Disposable Cannula (CAN/UCAN)

### INSTRUCTIONS FOR USE

Prepare LifePort Kidney Transporter and LifePort Kidney Transporter Perfusion Circuit as described in their Instructions for Use. Patches that contain multiple arteries may be able to be placed into one LifePort Kidney Transporter Disposable SealRing Cannula. Otherwise, refer to LifePort Kidney Transporter Disposable Coupler Instructions for Use to cannulate multiple vessels.

**CAUTION:** Where noted, perform the following procedure on an aseptic field using aseptic technique.

1. *Using standard aseptic technique*, introduce LifePort Kidney Transporter Disposable SealRing Cannula onto the sterile field.
2. *Using standard aseptic technique*, unhook the straps and open LifePort Kidney Transporter Disposable SealRing Cannula.
3. *Using standard aseptic technique*, slide the aortic patch through the center of the cannula ring.
4. *Using standard aseptic technique*, lay patch flat, making sure that the tissue covers the entire cannula ring. If necessary, instruments may be used to temporarily hold the tissue in place until LifePort Kidney Transporter Disposable SealRing Cannula is secure.
5. *Using standard aseptic technique*, close LifePort Kidney Transporter Disposable SealRing Cannula, securing the tissue between the two cannula halves.
6. *Using standard aseptic technique*, wrap each strap around both cannula halves and secure the straps to their posts.
7. *Using standard aseptic technique*, place the kidney in the Kidney Cradle and snap LifePort Kidney Transporter Disposable SealRing Cannula into the Cannula Mount.

**NOTE:** If perfusing multiple vessels, only the main vessel cannula shall be connected to the Cannula Mount.

8. *Using standard aseptic technique*, adjust the height of the Cannula Mount and rotation of the cannula to comfortably position the vessel.
9. *Using standard aseptic technique*, visually inspect the vessel, ensuring there are no twists or occlusions.
10. *Using standard aseptic technique*, secure the Mesh Organ Restraint over the kidney in the Kidney Cradle, allowing for slight swelling while being perfused.

**NOTE:** Follow the procedure in the LifePort Kidney Transporter Disposable Perfusion Circuit Instructions for Use on how to place the kidney and Kidney Cradle within LifePort Kidney Transporter Disposable Perfusion Circuit.

## Straight

CAN0003  
CAN0005  
CAN0008

### INDICATIONS FOR USE

LifePort Kidney Transporter (LKT) system is intended to be used for the continuous hypothermic machine perfusion of kidneys for the preservation, transportation and eventual transplantation into a recipient.

### DEVICE DESCRIPTION

LifePort Kidney Transporter Disposable Straight Cannula is used when the vessel to be perfused ends without a patch or when potential intimal lining damage is not a concern.

The complete LifePort Kidney Transporter system is comprised of the following components:

- LifePort Kidney Transporter (LKT100P/LKT101P)
- LifePort Kidney Transporter Disposable Perfusion Circuit (LKT201)
- LifePort Kidney Transporter Disposable Sterile Drape (LKT300)
- LifePort Kidney Transporter Disposable Cannula (CAN/UCAN)

### INSTRUCTIONS FOR USE

Prepare LifePort Kidney Transporter and LifePort Kidney Transporter Perfusion Circuit as described in their Instructions for Use. If cannulating multiple vessels, refer to LifePort Kidney Transporter Disposable Coupler Instructions for Use.

**CAUTION:** Where noted, perform the following procedure on an aseptic field using aseptic technique.

1. *Using standard aseptic technique*, introduce LifePort Kidney Transporter Disposable Straight Cannula onto the sterile field.
2. *Using standard aseptic technique*, insert LifePort Kidney Transporter Disposable Straight Cannula tip no further than necessary into the vessel. Minimizing the length of the cannula tip being inserted into the vessel will allow for resection of that segment of the vessel if necessary.
3. *Using standard aseptic technique*, secure the vessel in place using silk ties, vessel loops, or another appropriate material. A groove in the tip is provided for positioning and securing.
4. *Using standard aseptic technique*, place the kidney in the Kidney Cradle and snap LifePort Kidney Transporter Disposable Straight Cannula into the Cannula Mount.
5. *Using standard aseptic technique*, adjust the height of the Cannula Mount and rotation of the cannula to comfortably position the vessel.
6. *Using standard aseptic technique*, visually inspect the vessels and Coupler tubing, ensuring there are no twists or occlusions.
7. *Using standard aseptic technique*, secure the Mesh Organ Restraint over the kidney in the Kidney Cradle, allowing for slight swelling while being perfused.

**NOTE:** Follow the procedure in the LifePort Kidney Transporter Disposable Perfusion Circuit Instructions for Use on how to place the kidney and Kidney Cradle within LifePort Kidney Transporter Disposable Perfusion Circuit.

## Coupler

CAN1000

### INDICATIONS FOR USE

LifePort Kidney Transporter (LKT) system is intended to be used for the continuous hypothermic machine perfusion of kidneys for the preservation, transportation and eventual transplantation into a recipient.

### DEVICE DESCRIPTION

LifePort Kidney Transporter Disposable Coupler is used to connect two or more cannula when multiple vessels must be perfused.

The complete LifePort Kidney Transporter system is comprised of the following components:

- LifePort Kidney Transporter (LKT100P/LKT101P)
- LifePort Kidney Transporter Disposable Perfusion Circuit (LKT201)
- LifePort Kidney Transporter Disposable Sterile Drape (LKT300)
- LifePort Kidney Transporter Disposable Cannula (CAN/UCAN)

### INSTRUCTIONS FOR USE

Prepare LifePort Kidney Transporter and LifePort Kidney Transporter Perfusion Circuit as described in their Instructions for Use.

**CAUTION:** Where noted, perform the following procedure on an aseptic field using aseptic technique.

1. *Using standard aseptic technique*, introduce LifePort Kidney Transporter Disposable Coupler onto the sterile field.
2. *Using standard aseptic technique*, connect LifePort Kidney Transporter Disposable SealRing Cannula, Straight Cannula, or Universal SealRing Cannula to each of the vessels to be perfused according to their Instructions for Use.
3. *Using standard aseptic technique*, identify a main vessel.
4. *Using standard aseptic technique*, place the kidney in the Kidney Cradle and snap the cannula connected to the main vessel into the Cannula Mount.
5. *Using standard aseptic technique*, adjust the height of the Cannula Mount and rotation of the cannula to comfortably position the vessel.
6. *Using standard aseptic technique*, replace the End Cap with one end of the Coupler.
7. *Using standard aseptic technique*, attach the second end of the Coupler to the port of the next cannula.
8. *Using standard aseptic technique*, use an additional Coupler and repeat steps 6 and 7 for each LifePort Kidney Transporter Disposable Cannula being used.
9. *Using standard aseptic technique*, visually inspect the vessels and Coupler tubing, ensuring there are no twists or occlusions.
10. *Using standard aseptic technique*, secure the Mesh Organ Restraint over the kidney in the Kidney Cradle, allowing for slight swelling while being perfused.

**NOTE:** Follow the procedure in the LifePort Kidney Transporter Disposable Perfusion Circuit Instructions for Use on how to place the kidney and Kidney Cradle within LifePort Kidney Transporter Disposable Perfusion Circuit.

**Storage Conditions:** Store between 2°C and 40°C. Avoid excessive heat and humidity. Keep dry and keep away from direct sunlight. Sterile unless package is damaged or open.

**Technical Assistance:** Please contact Organ Recovery Systems 24/7 Perfusion Helpline at +866.682.4800 (toll free in US), +32.2.715.0005 (Belgium), +55 11 9 8638 0086 (Brazil), +33.967.23.00.16 (France), or +34.910.91.16.16 (Spain).

**Contraindications:** There are no known contraindications when used as directed.

**WARNINGS AND CAUTIONS**

- R<sub>Only</sub>** **CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

---

-  **CAUTION:** LifePort Kidney Transporter Disposable Products should be stored indoors in a dry location out of direct sunlight.

---

-  **WARNING:** Visually check LifePort Kidney Transporter Disposable Products. Do not use if parts are loose, cracked, broken, or liquid is leaking. Check that all fittings are tight before installation.

---

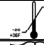
-  **WARNING:** For single use only. Do not reuse, reprocess or resterilize. Reusing, reprocessing, or reesterilization of single-use devices creates a potential risk of patient or user infections due to contamination. This contamination may lead to injury, illness, or other serious patient complications.

---


-  **WARNING:** Use standard aseptic technique and universal precautions (e.g., gloves, masks, gowns, goggles or equivalent eye protection, biohazard bags) when handling the kidney, and when handling and disposing of Life-Port Kidney Transporter Disposable Products and perfusate to prevent the possible transmission of pathogens to medical personnel and patients. The single practitioner working alone must pay special attention to maintain these conditions.

---

**EXPLANATION OF SYMBOLS**

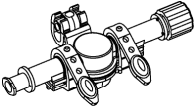	Warning/Caution		Use By, YYYY-MM-DD		Temperature Limits
<b>LOT</b>	Lot Number		Date of Manufacture, YYYY-MM-DD		Consult Instruction for Use
<b>REF</b>	Reference Number		Manufacturer		Keep Away From Sunlight
	Do Not Reuse		Do Not Restertilize		Keep Dry
<b>STERILEEO</b>	Sterile Medical Devices Using Ethylene Oxide			<b>R<sub>Only</sub></b>	Prescription Medical Device

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